## SNORE GONE (MC0099)

PLS READ THIS INSTRUCTION CAREFULLY BEFORE USING SNORE GONE WARNINGS

- 1. NOT FOR USE DURING PREGNANCY
- 2. PERSONS WITH HEART PROBLEMS: DO NOT USE.
- 3. PERSONS WITH SUSPECTED OR DIAGNOSED EPILEPSY: DO NOT USE.
- 4. SNORE GONE MUST NOT BE USED OVER INFECTED OR INFLAMED AREAS OR SKIN ERUPTIONS,
- 5. SNORE GONE MUST BE KEPT OUT OF REACH OF CHILDREN.

**SNORE GONE** designed with an adjustable Velcro strap which you can put on your wrist like a watch Which is absolutely safe, painless and with no side effect to the user.

## HOWITWORKS

SNORE GONE with 2 electrode pad at the back of the unit and once it contact to the skin, It can detect the snoring from the user and will deliver a gentle electrical pulses for few seconds, in order to make user to change his sleeping position without wake you up.

## BATTERY INSTALLATION

Insert a CR2032 battery into the battery compartment by sliding the battery door at the back side of the **SNORE GONE** Remember, the Positive "+" side of the battery should be face upward. Then close the battery door.

## HOW TO OPERATE

- 1. Simply apply the conducting gel on your wrist (the area for contacting with the electrode pad of **SNORE GONE**)
- 2. Then put on the SNORE GONE on your wrist with the Velcro strap to adjust the size.
- 3. To turn on the unit by press "Power/Start" button and hold until green LED flashes.
- 4. Press "Max" or "Min" button to adjust Intensity to select your comfortable level.
- 5. To start the programmer, press "Power/Start" button once again, and you would find the green LED still flashing until the LED stop flashing, then the product is on working
- 6. So once it can detect the snore, it will deliver a gentle electrical pulses to the user in order to make him change his sleeping position
- 7. To tum off the unit, press the "Power/Start" buttonand hold until red LED flashes.

