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Digital display color screen alcohol tester
(Home Edition)

In order for you to better use this product, please read the instructions carefully and strictly follow the instructions before use. Everyone has different reactions to alcohol, and the test results are for reference only and cannot be used as a criterion for judging whether to drink or drive. Manufacturers and sellers will not bear any legal responsibility for illegal activities.

Packaging list:
1. Alcohol tester 1 PCS
2. Charging cable 1PCS
3. User Manual 1PCS

Key and interface distribution:

(1) Blowing nozzle (2) Tester life indicator light (3) Display screen
(4) Test results (5) Key on (6) Charging port

Drinking and driving: refers to the driving behavior of vehicle drivers whose blood alcohol content is greater than or equal to 20mg/100ml and less than 80mg/100ml.
Drunken driving: refers to the driving behavior where the alcohol content in the blood of the vehicle driver is greater than or equal to 80mg/100ml.

Function description:
a. Display of numbers and warning bars, accompanied by audible alarms;
b. Rapid response and recovery;
c. Reminder of sensor service life.
d. Automatic shutdown and low voltage prompt;
e. 50 sets of data records. (In the power-on state, press and hold the button for two

seconds to view the recorded data. The "Record" character is displayed, displaying the data serial number and corresponding concentration value. Short press the button to scroll up. If there is no action within 10 seconds, the display will flash OFF twice to shut down.)

Sensor lifespan:
500 test cycles are green dots, 501-1000 are yellow dots, and 1000 or more are red dots.
After the number of uses reaches 2000, the red dot flashes, indicating that the sensor's lifespan is approaching its limit and the function is functioning normally. Users can choose to continue using or replace the tester.
After reaching 2500 times, only the sensor lifespan character will be displayed, and after the red light flashes twice, it will display OFF flashing twice to shut down. This is displayed every time the machine is turned on, and the tester is no longer in use.

Low battery prompt:
When the voltage is greater than 3.7V, it displays full scale, while when the voltage is 3.2-3.6V, it displays half scale. When the voltage is lower than 3.2V, the half scale symbol of the battery flashes, prompting the user to charge. When charging, the half cell symbol in the battery box flashes, and when fully charged, the full cell is displayed. When the battery level is too low at 3.1V, the power on battery symbol flashes twice, and then displays OFF flashing twice to shut down.

usage method:
preheat:
1: Turn on the machine. Press and hold the button for 2 seconds, the buzzer "B" will sound and the LCD will display full screen for 1 second, "Ready" display, while countdown "10..09..01".
2: After the countdown ends, the buzzer "B" sounds, and the LCD displays "Start" while the countdown is "10..09..01", blow at any time during this countdown, and immediately reverse after sensing gasNumber "03-02-01", buzzer B will sound to display the result, and the "result" character will be displayed The results are as follows:
(1) < 0.15mg/L, triangle pointing green
(2) 0.15mg/L - 0.25mg/L, Triangle pointing yellow, buzzer BB... BB... short sound alarm
(3) > 0.25mg/L, triangular symbol points to red, buzzer BBB... urgent sound alarm
3: After displaying the results for 10 seconds, "OFF" flashes twice before shutting down.

be careful.
Due to oxidation on the sensor surface when the tester is not in use for a long time, users need to preheat the product as many times as possible before conducting an alcohol test when removing the tester from its packaging for the first time or when the tester is not in use for a long time.

matters needing attention:
1: Perform the test after drinking for at least 20 minutes as alcohol is absorbed into the bloodstream from the digestive system. It takes about 20 minutes in the liquid. Immediately test after drinking, the results can only reflect the oral cavity at that time. The alcohol concentration, not the alcohol concentration in the blood.
2: To ensure accurate test results, please wait for 20 minutes after drinking alcohol before conducting the test. The testing time is the most it has been more than 3 minutes since the last test.
3: Do not directly wipe the product with chemicals or corrosive solvents to avoid liquid. The body flows into the air blowing port.
4: When there is a high concentration of interfering gas present, the machine will not function properly.
5: If tested under low battery conditions, there will be a certain deviation between the test value and the actual value.
6: It is strictly prohibited to operate in sealed environments with corrosive gases such as pigments, insecticides, alcohol, or air pollution. Placement and use.
7: Unauthorized repair and damage to internal parts can lead to product malfunction.
8: Prevent dangerous actions such as falling, squeezing, and severe vibration from a height.
9: In order to achieve more accurate test results, it is recommended that users repeat the test 2-3 times.

Declaration:
1: Regardless of whether the product is used or not, manufacturers and sellers will not bear any responsibility for illegal activities. Responsibility. For example: drunk driving, high-altitude work, etc.
2: Everyone has different reactions to alcohol, so the test results are for reference only and cannot be used as a decision Criteria for determining results.

Alcohol Concentration Comparison Table

order number	BAC Blood alcohol concentration				BrAC Alcohol concentration in exhalation	
	mg/100ml	%	‰	g/l	mg/l	common ppm
1	10	0.01	0.1	0.1	0.05	25
2	20	0.02	0.2	0.2	0.10	50
3	30	0.03	0.3	0.3	0.15	75
4	40	0.04	0.4	0.4	0.20	100
5	50	0.05	0.5	0.5	0.25	125
6	60	0.06	0.6	0.6	0.30	150
7	70	0.07	0.7	0.7	0.35	175
8	80	0.08	0.8	0.8	0.40	200
9	90	0.09	0.9	0.9	0.45	225
10	100	0.10	1.0	1.0	0.50	250
11	110	0.11	1.1	1.1	0.55	275
12	120	0.12	1.2	1.2	0.60	300
13	130	0.13	1.3	1.3	0.65	325
14	140	0.14	1.4	1.4	0.70	350
15	150	0.15	1.5	1.5	0.75	375
16	160	0.16	1.6	1.6	0.80	400
17	170	0.17	1.7	1.7	0.85	425
18	180	0.18	1.8	1.8	0.90	450
19	190	0.19	1.9	1.9	0.95	475
20	200	0.20	2.0	2.0	1.00	500

BAC: Blood Alcohol Concentration (Blood alcohol concentration)
BrAC: Breath Alcohol Concentration (Expiratory alcohol concentration)
0.05%BAC: It refers to the presence of 0.05g (50mg) of alcohol per 100ml of blood

Sensor

high sensitivity direct heating alcohol sensor

Test range	0.00-0.9mg/l
Charging voltage	DC3.7V
Preheating time	10 seconds
Blowing time	10 seconds
Reaction time	≤ 5 seconds
Automatic shutdown	10 seconds after displaying the results
Product size	58.74 * 30.6 * 13.5mm
Working temperature	0-35 ℃
Storage temperature	-5~50 ℃

Common faults and solutions:

Fault phenomenon	Possible fault causes	Solution
Unable to start the machine	The battery is severely low Circuit fault	Please charge the product Contact after-sales service for handling
Blowing without reaction	Not fully preheated Circuit fault	Warm up the product several times before conducting testing Contact after-sales service for handling
No result display for blowing	There is residual gas in the product Failure to blow within the specified time	Reheating the product Blow air within the displayed blowing character time on the display screen
Result display without blowing	There is residual gas in the product	Warm up a few more times before testing
Several different test results	Test immediately after drinking alcohol The blowing time is different	20 minutes after drinking before testing Suggest deep breathing and blowing for 3-5 seconds

Units commonly used in various countries:
mg/100ml - China %BAC - America g/l - Europe mg/L BrAC - Japan